

US Shipping Instructions - to ATLANTA

Important: Retain the shaded, left-hand side of the Billable Stamp.

This is your customer receipt for tracking purposes.

- 1 Write** the date, your name, address, and telephone number in the **“From”** section on **the right-hand side** of the FedEx® Billable Stamp (shipping label). **DO NOT** make any other changes to the stamp or sign the **“Release Signature”** area.
- 2 Peel** off the adhesive backing on the FedEx Billable Stamp.
- 3 Adhere** FedEx Billable Stamp onto the FedEx Express Clinical Lab Pak mailer. **Please Note:** If your kit contains a FedEx Billable Stamp that is **purple** on the left side, your sample requires an overnight shipment. Please ship your specimen **Monday - Friday only**. If your kit contains a FedEx Billable Stamp that is **gray** on the left side, your sample can ship over the weekend.
- 4 Place** the specimen collection kit box into the FedEx Clinical Lab Pak mailer and seal.
- 5** If possible, **drop-off** the package at the closest FedEx Drop-Off Center. If no Center is available to you, please contact 800.463.3339 to schedule a pickup.

Please Note

Holiday shipping:

- **Purple** billable stamps: **Do not ship 1 business day** before a national holiday.
- **Gray** billable stamps: **Do not ship 2 business days** before a national holiday.
- Ship to:

Genova Diagnostics
3425 Corporate Way
Duluth, GA 30096, USA

All tests must be ordered by healthcare professionals only.

Please contact your healthcare professional for lab results.



Call 800.522.4762 or visit our website at www.gdx.net