

ENSURE THE FOLLOWING:

- Peel and stick labels completed** with **patient's name** and **date of collection** are on all tubes as well as the test requisition form

All tubes:

- Are tightly closed
- Sealed in the biohazard bag with absorbent pad
- Refrigerated until packaged for shipping

All required information:

- All sections of **test requisition form** completed
- Payment information provided**
- All tubes and associated forms** placed back in the original Genova sample collection pack box prior to shipping

SHIP THE SAMPLE(S) TO THE LAB

Ship only Monday through Thursday, and within 24 hours after final collection.

Please refer to the shipping instruction insert found in your Genova sample collection pack box.



Call +44 (0)20 8336 7750 or visit our website at www.gdx.net/uk

Stool Culture

GASTROINTESTINAL

1 DAY COLLECTION

PATIENT SAMPLE COLLECTION INSTRUCTIONS FOR THE FOLLOWING PROFILE(S)

H. pylori Stool Antigen HpSA	Stool	#DIG11
Microbiology Analysis	Stool	#DIG07
Yeast Culture with KOH prep	Stool	#DIG18

COLLECTION MATERIALS FOR SAMPLE



ORANGE-TOP
C & S TUBE



FECAL SWAB
AND TUBE



COLLECTION
DEVICE

- **CAUTION: Tubes contain poisonous liquid. KEEP OUT OF REACH OF CHILDREN.**
- Tubes are under pressure. Cover tube cap with a cloth and remove cap slowly.
- For eye contact, flush with water for 15 mins.
- For skin contact, wash with soap and water.
- For ingestion, contact poison control center immediately.

REQUIRED MATERIALS

- Disposable glove
- Peel and stick labels
- Absorbent pads
- Test requisition form
- Biohazard bags
- Genova sample collection pack box

IMPORTANT INFORMATION BEFORE YOU BEGIN THE COLLECTION



- Test not recommended for patients under 2 years of age.
 - **Wait at least 4 Weeks** from colonoscopy or barium enema before starting the test.
 - Please consult with your physician before stopping any medications. Certain medications and/or supplements may impact test results.
 - **2 to 4 Weeks Before the Test:**
 - » Discontinue antibiotics, antiparasitics, antifungals, probiotic supplements (acidophilus, etc.).
 - » Discontinue proton pump inhibitors (PPIs), and bismuth **14 Days prior if adding on the H. pylori test.**
 - **2 Days Before the Test:**
 - » Discontinue aspirin and other NSAIDs (i.e. ibuprofen), rectal suppositories, enemas, activated charcoal, bismuth, betaine HCL, digestive enzymes, antacids, laxatives, mineral oil, castor oil, and/or bentonite clay.
 - **DO NOT collect samples** when there is active bleeding from hemorrhoids or menstruation.
 - Before collecting your specimen refer to the shipping instruction to determine what day you can ship.
- Ship only Monday through Thursday, and within 24 hours after final collection.**

COLLECTION

- 1 **Completely fill out** front and back of test requisition form using the included form.

STOOL COLLECTION


- 2 **Put on** the glove.
- 3 **Collect** your stool sample using the enclosed collection container. **DO NOT contaminate** the sample with either urine or water from the toilet.
- 4 **ORANGE-TOP TUBE:** Remove the cap. **Transfer** stool sample into the tube using the built-in scoop. **Collect** from different areas of the sample. **Mix** the sample with the liquid in the tube until it is as smooth as possible. **Make** sure that the liquid and sample do not exceed the **FILL LINE**. **DO NOT OVERFILL**. **Screw** the cap on tightly. **Shake** tube for 30 seconds.

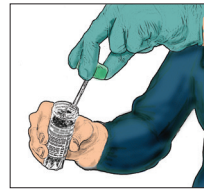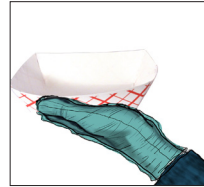

**BLENDED SAMPLE
& PRESERVATIVE
CANNOT EXCEED
THE RED FILL LINE**

- 5 **Peel** open swab package, **remove** the tube, and place it upright. The swab should remain in the sleeve until you are ready to collect sample.

- 6 **Grasp** swab above the molded breakpoint which is the opposite end from the nylon applicator tip. (see diagram below)

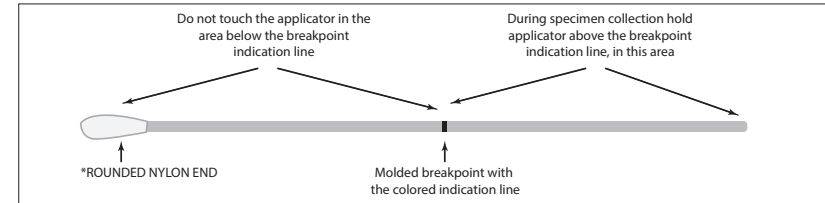

- 7 **Collect** sample by inserting the **ROUNDED NYLON END*** (see above) of the swab into the stool sample and **rotate** it. **Confirm** that the swab contains fecal material. If not, repeat.

- 8 **Open** the swab collection tube and insert the swab. **Mash** and **mix** the rounded nylon end of the swab with stool on it against the side of the tube.

- 9 **Break** the swab off at the break point. **Place** the screw cap on the tube and **tighten**. **Shake** the tube. Using the peel and stick label, **write** patient's date of birth on the label and apply to the swab tube.

- 10 **Dispose of remaining sample** into toilet and dispose of collection container and glove.

- 11 **Place** all tubes in the biohazard bag and refrigerate. **Refrigerate** until ready to ship. **DO NOT FREEZE**.

